Thank you for choosing Juzo, an international leader in medical compression garments since 1912.

Manufactured with care and detail, your Juzo Compression Wrap has several features that provide therapy, comfort, versatility and superior fit, including:

- Working pressure - Short stretch for a high working pressure.
- Gap resistant - Special notches and minimal overlying straps resist gapping and bulk.
- EZ pull tab - Aid for hassle free application.
- Reversible - Worn in either beige or black

Included In This Package:

- Juzo Compression Wrap
- Juzo Hook Fasteners

Care and Maintenance

- Remove all the half circle hook fasteners from the straps
- Machine wash warm
- A garment bag is strongly recommended
- Tumble dry low
- DO NOT USE: bleach, chlorine, fabric softeners, or other laundry additives
- Store garments in a dry place at room temperature

Return Policy

Should it be necessary to return your garment for evaluation, Juzo requires that the garment be laundered (for hygienic reasons) prior to returning to the original place of purchase.

Fabric

100 % Nylon / 100% Neoprene
Not made with natural rubber latex.

Instructions for Use

It is recommended that you consult with your medical professional and fitter for proper guidelines on the use and application of this product. If you experience any discomfort, immediately remove the short-stretch compression wrap and consult with your medical professional.

- Juzo Compression Wrap is designed to be reversible. The half circle hook fasteners need to be attached to the garment on the opposing side of the color to be worn.

Indications

Primary & secondary varices, chronic venous insufficiency, post-thrombotic syndrome, venous stasis ulcers, pre and post-vein procedure and lymphatic edema.

Contraindications

Absolute: peripheral arterial disease, decompensated heart diseases, septic phlebitis, phlegmasia coerulea dolens.

Relative: Leg infection, intolerance to compression or garment fabric, sensory disturbance to the limb.

Cautions

No side effects are currently known. However, if a negative result should occur (i.e. skin irritations) from the use of our products, please discontinue use of the garment and consult with a medical professional. The manufacturer is not liable for damage/injuries which have been caused by improper handling or misuse.

Steps to Putting on the Thigh Wrap

1. The Thigh wrap should be donned after putting the Knee Wrap segment on the leg.
2. Attach the hook fasteners to the straps on the side opposite of the color to be worn. For example, if beige is the desired color, then the fasteners should be applied to the black side of the garment.
3. Gently fold the lower two fasteners over onto themselves to prevent the fasteners from inadvertently attaching to the garment. Each fastener can then be unfolded before attaching.
4. While in the standing position the bottom of the thigh wrap should overlap the top portion of the knee wrap.
5. Secure the top strap of the thigh wrap by counter pulling the top strap and the top EZ Pull at the same time. The strap should be just tight enough to gently hug the thigh and hold the garment in place.
6. Fasten the middle strap by simultaneously counter pulling the middle and lower strap until the middle strap gently hugs the leg.
7. Fasten the lower strap by simultaneously counter pulling the lower strap and the EZ Pull until the strap gently hugs the leg.
8. Repeat steps 5-7 pulling each strap until they are comfortably snug. Each strap should slightly overlap the strap above it using the notches built into the end of the strap nearest the main body of the garment.
9. The wrap should have a snug fit but not be painfully tight. If you feel any pain or discomfort or the wrap feels too loose, then repeat steps 6 through 8, adjusting the level of tension that is applied.

Note: See care and application directions for use of Juzo Compression Wrap Knee, Calf & Foot Segment.
Extra fasteners are provided inside packaging.

JUZO COMPRESSION WRAP

Short Stretch Compression Wrap

Venda de compresión de elasticidad reducida
Bandage compressif à extensibilité courte

- High Working Pressure
- EZ Pull Tabs

REVERSIBLE


beige or black garment

Freedom in Motion

For over 100 years, our family philosophy has been that medical garments should not inhibit a person's lifestyle.
Venda de compresión de elasticidad reducida para piernas saludables y siempre en movimiento.
Bandage compressif à extensibilité courte pour aider à améliorer la circulation et garder les jambes en bonne santé et en mouvement.
Please Recycle
Printed in USA
INTHIGHWRAP-0871

GARMENT CONTENTS & CARE
See inside of insert for material contents & proper care.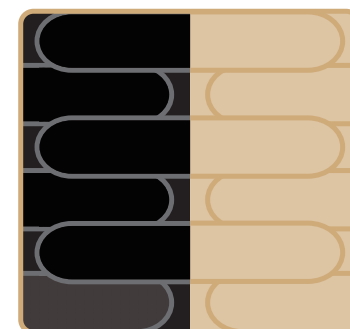

REVERSIBLE
beige garment or black garment

Manufactured By
Julus Zorn, Inc • PO Box 1088
Cuyahoga Falls, OH 44223
Distributed by Juzo USA
www.juzousa.com
Distributed by Juzo Canada LTD.
www.juzo.ca

The Juzo Compression Wrap will offer a varying degree of pressure that is dependent on the tension of the garment and the position and activity level of the wearer.

Size Chart For Juzo® Compression Thigh Wrap
Note: This size chart is not intended to replace a professional fitting by a qualified fitter.

	S	M	L	XL	2XL
F	35-55 cm 13 3/4"-21 3/4"	48-70 cm 19"-27 1/2"	60-82 cm 23 1/2"-32 1/4"	70-92 cm 27 1/2"-36 1/4"	80-102 cm 31 1/2"-40 1/4"
G	43-70 cm 17"-27 1/2"	54-81 cm 21 1/8"-32"	65-92 cm 25 1/4"-36 1/4"	75-102 cm 29 1/2"-40 1/4"	85-112 cm 33 1/2"-44 1/4"

psz-01-56

- High Working Pressure: Short stretch for a high working pressure
- Gap Resistant: Special notches and minimal overlying straps resist gapping and bulk
- EZ Pull Tabs: Pull tab aid for hassle-free application
- Made in the USA: Juzo Compression Wraps are made in the U.S.A.

The Juzo CompressionWrap™ is an effective and easier-to-use alternative to bandaging. The versatile wrap delivers therapeutic compression and increased working pressure to improve your quality of life.

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