upright walker
andador vertical
déambulateur

item #s:

UPWKRBL10

UPWKRGY10

TM
drive

www.drivemedical.com

parts list

WEIGHT CAPACITY: 300 LBS (136 KG)

- | | |
|--------------------------|-------------------------|
| A. Hand Brake | G. Push Handles |
| B. Backrest | H. Padded Armrests |
| C. Seat | I. Adjustment Knob |
| D. Removable Storage Bag | J. Cane Holder |
| E. 10" Front Wheels | K. Sit to Stand Handles |
| F. 8" Rear Wheels | |

introduction

The Upright Walker is designed to increase stability by helping users prevent slouching and hunching. It helps promote better posture as you are walking to create stable and comfortable mobility.

upright walker includes:

1. Removable Storage Bag
2. Backrest
3. Operating Instructions
4. Front Casters
5. Arm pads with tube stand
6. Cane Holder

IMPORTANT SAFETY INFORMATION

- Please avoid using this product if you suffer from
 - Visual impairments
 - Balance issues
 - Certain disabilities that effect the disability of arms, shoulders, or hands
- Please use as a walking aid, and not as a transportation device as it is not meant to be moved while anyone is sitting on it.
- Always make sure that all parts are secure before use.
- Always lock the brakes before sitting down and make sure the walker's loop locks are in the engaged in the down locked position.

WARNINGS

- Please make sure the walker is fully opened, seat is in a full open position, and the folding bar underneath the seat is locked in a down position.
- The walker should be used on a flat surface. Ensure that you are accompanied by another person when first using the walker.
- When using the walker please make sure your hands are always near the push handle to brake when needed.
- Avoid sudden braking on wet or smooth surfaces
- It is not recommended to use the walker in poor weather (rain, snow, sleet, etc.)
- When heading downhill please slow down
- Make sure brake cable is running inside of the side frame
- Maximum user weight: 300 lbs.

- Regularly check the screws and fasteners on the Upright Walker and retighten if necessary.
- Use the removable portable bag only for transport of objects and do not overload. Maximum load: 11 lbs.
- Do not expose the product to temperatures above 104°F for an extended period of time, as this can cause the Upright Walker parts to overheat and burn the user.
- Be sure that both knobs are fully tightened and are at equal heights. Test Walker for stability before applying full weight to hand grips.
- Replace tires when significantly worn or damaged.
- **DO NOT MOVE WALKER WHILE SEATED.** This is a walking aid only and is not to be used as a transportation device.

assembly Instructions

Follow the instructions below to properly assemble your upright walker.

Insert front wheels into receptacles on the front of frame.

Unfold frame upwards until it is perpendicular to wheel base.

Open frame outwards and press seat down until the seat is flat.

Insert the backrest into upright handle tube and secure by turning knob clockwise. Note, flat side of backrest should face downward.


Insert the upright handle tube into the walker column, and adjust the height to the preferred user setting using light gray push button.

Insert the push handle into the steering column.


assembly Instructions

7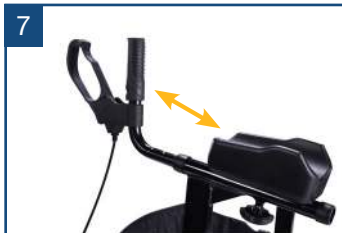

Adjust to a comfortable length and secure the knob in place by turning it clockwise.

8

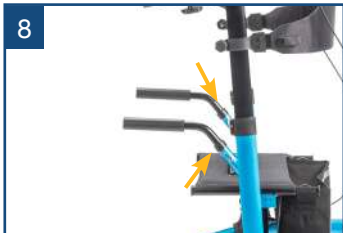Insert each sit-to-stand handle into the frame. Ensure push buttons click into place.

9

To attach storage bag, pull out the red reflectors located on the front of the seat frame.

10

Place the storage bag onto the walker making sure the bag is positioned evenly.

unfolding the Upright Walker

To unfold the Upright Walker pull the handles upward from the full knocked down position. Once walker is at a 90 degree angle, position the Walker on all four wheels. Hold each hand grip and pull outward until the Walker unlocks and opens fully

(figure 11). Gently push down on each side of the seat rails near the vertical frame to ensure the unit is fully opened. An audible 'click' confirms that the latches are secure and the seat is fully engaged in a locked position (figure 12).

11

12

height adjustment

To adjust the handle height, press the height adjustment button and pull the handgrip upwards at the same time (figure 13). Release the button once the desired height is reached. An audible 'click' confirms the handle tube is secure in the locked position. Repeat this process until the desired height is reached for both handles. Refer to the numbering system (figure 13) to ensure both handles are adjusted to the same height.

backrest adjustment

To adjust backrest height slide backrest to desired height then turn knob clockwise to tighten. (figure 14)

removing/attaching the storage bag

When removing or attaching the storage bag located on the end of the seat frame, pull the front knobs outwards. The knobs are in a fixed position, held in place with screws. Unscrew the screws in order for the holders to telescope out or in. Once in your desired position, re-fasten the screws. Use the two side straps on the carry on bag to gently slide the bag on or off (Figure 15 & 16). To remove bag, fold the Upright Walker slightly and reverse above steps.


braking/loop lock operation

To slow down or come to a temporary stop. Pull back slowly on the loop locks with four fingers until unit slows down or comes to a temporary stop. To lock wheels in position.

Press forward on both loop locks with palms until you hear a "click" that will indicate wheels are locked.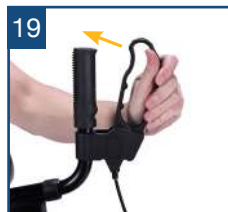

To release wheel locks. Pull back both loop locks with palms of hand to original position.

adjustment of brakes

The hand brake system fitted on this walker complies with the manufacturer's quality standards. The hand brake system has been preset (adjusted) to the best performance settings. To better suit you personally, please follow the instructions set out below. You may need to adjust the hand brake systems from time to time as they will wear out with normal use.

20

Hold firmly in place do not turn.

To tighten the hand brake adjustment. "turn the nut clockwise"

To loosen the hand brake adjustment. "turn the nut counter clockwise"

sitting

Position the open and locked upright walker on a flat, even, solid surface on all four wheels. Lock the brakes. The upright walker must remain upright and must not move. Turn around to sit on the seat. Sit straight up and do not lean backwards.

**DO NOT MOVE WALKER WHILE SEATED.
THE BRAKES MUST REMAIN LOCKED.**

folding / transport / storage

To fold, pull the seat upwards by the handle located in the middle of the seat as shown in figure 21. The walker can now be folded completely and secured by pushing the hand grips together. The arms then fold down toward the frame into the full folded position (figure 22).

curb assist

The Upright Walker features a Curb Assist for terrain transitions. The Curb Assist allows the user to move over curbs and small steps with ease. The Curb Assist is visibly located on the inside of the rear wheel (figure 23).

how to use the curb assist:

1. When approaching a curb, step or transition firmly grasp both hand grips and place your right foot on the Curb Assist located on the right-hand side of the walker. Be sure to use your right foot to activate the Curb Assist. DO NOT use your opposite foot.
2. While securely grasping both hand grips, gently apply pressure to the Curb Assist with your foot
3. Bring the front wheels of the walker to a slight angled position just above the curb or step pushing forward if necessary
4. Once at a stable position, gently release your foot from the Curb Assist allowing the front wheels to rest of the curb, step or transition
5. Push the Walker forward as the rear wheels approach the curb, step or transition and lift the rear wheels onto the curb, step or transition
6. Proceed with walking once observing that all four wheels are safely past the curb, step or transition

cleaning

Clean regularly by hand with a damp cloth. Do not use aggressive detergents. Drive recommends periodic visual inspection of this product by the user, to ensure that all parts and hardware are secure, that components are in good working order and not worn, torn, frayed, or loose, and that there are no obstructions or impediments to normal, safe operation. If any part or mechanism appears to be unsafe or compromised, do not use the product. If you experience problems, please contact the provider that sold you the product or visit www.drivemedical.com/help for more info.

maintenance

Drive recommends periodic visual inspection of this product by the user, to ensure that all parts and hardware are secure, that components are in good working order and not worn, torn, frayed, or loose, and that there are no obstructions or impediments to normal, safe operation. If any part or mechanism appears to be unsafe or compromised, do not use the product. If you experience problems, please contact the provider that sold you the product or visit www.drivemedical.com/help for more info.

lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

 **WARNING:** This product can expose you to chemicals, including bisphenol A (BPA), which is known to the State of California to cause birth defects or other reproductive harm. For more information go to www.P65Warnings.ca.gov.

© 2024 Medical Depot, Inc. All rights reserved.

Drive is a trademark of Medical Depot, Inc.

Port Washington N.Y. 11050 USA Made in China

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© 2024 Medical Depot, Inc. Tous droits réservés.
Drive est une marque de commerce de Medical Depot, Inc.