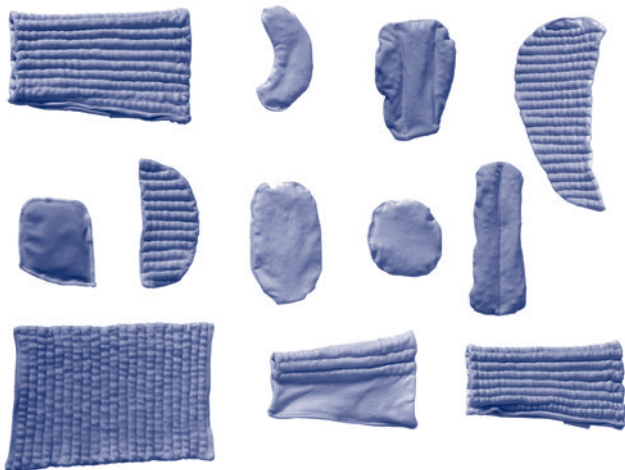
JOBST®

INSTRUCTIONS FOR USE INSTRUCTIONS D'UTILISATION

JOBST® PitPak

Ready-to-wear cherry pit-filled pads

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INSTRUCTIONS FOR USE

Thank you for choosing JOBST® PitPaks, our cherry pit-filled pads, providing multiple pressure points against deep and thick fibrosis in lymphedematous tissue, complementing daily lymphedema treatment.

1. Product description

JOBST® PitPaks are designed to locally increase pressure for patients with hard fibrotic tissue due to lymphedema. The cherry pits create multiple pressure points when used under compression and are designed to stimulate lymph flow and soften fibrotic tissue.

2. Intended use and indications

JOBST® PitPaks are indicated for hard and dense fibrotic tissue of patients with lymphedema (primary or secondary lymphedema) in conjunction with compression.

Please contact your healthcare professional for appropriate product selection and application.

3. Material composition JOBST® PitPak

Fabric: 91% organic cotton, 9% elastane (spandex)
Filling: 100% cherry pits

4. Directions for use

- Position the JOBST® PitPak on the affected area and apply gentle pressure, carefully avoiding wrong positioning of the pad. Avoid too high pressure built-up.
- Begin with an application of max. 15 minutes under light compression to assess the situation.

Closely monitor the condition of the skin and correct positioning of the pad, when extending the application.

Special notes:

- Consult your healthcare professional for correct selection, application and use of JOBST® PitPaks.
- Long fingernails and jewelry may damage the product.
- Do not bring the product into contact with greasy ointments, body lotions or similar substances.
- Loose threads or loops should not be cut off since this may cause holes or runs to form in the product.
- Intended to be reused by a single patient only.

5. Contraindications

JOBST® PitPak should not be used in case of:

- Patients with soft or pitting edema.
- Patients with open wounds in the affected area.
- Impaired sensitivity of the area, peripheral nerve damage.
- Known sensitivity to any of the product materials.

If used as part of a compression device, contraindications or cautions linked to the compression application are applicable for the complete compression device.

6. Warnings and precautions

Please consider, if JOBST® PitPak is applicable in case of:

- Weeping dermatoses
- JOBST® PitPak should only be used after consultation with your healthcare professional and be checked regularly by them on the appropriateness of your therapy, also considering age and health status.
- This can be an aggressive form of therapy that should be carefully monitored by a highly trained lymphedema therapist.
- Application of JOBST® PitPak over bony areas for an extended time has to be closely monitored.
- If you notice discomfort, pain, numbness, changes in skin color, appearance of new wounds or swelling during application, stop using the product and contact your healthcare professional.
- Appropriate hygiene and skin care should be applied.

7. Storage and/or special handling conditions that apply

Please protect your product from direct exposure to sunlight, heat, and moisture.

We recommend storage at room temperature.

Due to wear or slackening of the material, the medical efficacy can be guaranteed for a wearing period of 6 months for the JOBST® PitPak.

This assumes correct handling (e.g. product care).

8. Notice

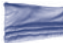
Any serious incident that has occurred in relation to this device should be reported to BSN medical Inc. (BSN medical GmbH within the European Union) and the competent authority of your state.

INSTRUCTIONS FOR USE

9. Size table

Patient measurements should be taken by a trained person to determine correct product fit.

JOBST® PitPak	Description	Size	Dimensions
	Rectangle Versatile shape for multiple uses	One size	15 x 23 cm (stitched rows)
	Tapered strip e.g. for the lateral forearm	One size	6 x 18 cm (2 channels with loose fill)
	Dorsum e.g. for the dorsum of the hand or foot	One size	10 x 13 cm (3 channels with loose fill)
	Round Versatile shape for multiple uses, e.g. in small areas with fibrotic tissue	Small Large	10 cm Ø (loose fill) 15 cm Ø (3 channels with loose fill)
	Oval e.g. for the ulnar aspects of the forearm	Small Large	10 x 15 cm (loose fill) 14 x 22 cm (3 channels with loose fill)
	Crescent e.g. for base of toes or inferior aspect of the breast	XSmall Small	4 x 10 cm (loose fill) 5 x 13 cm (3 channels with loose fill)
	Half circle e.g. sub-mandibular region	Small Medium	15 cm length 20 cm length
	Inframammary e.g. inferior aspect of the breast	Small Medium Large XLarge	10 x 22 cm (stitched rows) 11 x 23 cm (stitched rows) 13 x 24 cm (stitched rows) 14 x 25 cm (stitched rows)
	Full channeled cylinder e.g. circumferential use on arm or leg	Small Medium Large XLarge	18 - 23 cm circumference, 20 cm length 23 - 30 cm circumference, 20 cm length 28 - 38 cm circumference, 28 cm length 36 - 43 cm circumference, 28 cm length
	Half channeled cylinder e.g. arm or leg	Small Medium	18 - 25 cm circumference, 19 cm length 20 - 30 cm circumference, 22 cm length

10. Cleaning and care

Additional information:

- Hand wash is recommended
- Wash once a week
- Wash when soiled

- Do not add fabric softeners
- Do not wring or twist
- Dry away from direct heat
- Wash with similar colors

Brand	Washing	Bleaching	Drying	Ironing	Professional Cleaning
JOBST® PitPak					

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BSN medical (Aust.) Pty Ltd